


THERMACRYL BEADS

FOR CHAIRSIDE REALIGNMENT

1. Place beads in a cup or bowl of hot water, and heat to 180 degrees or above.
2. Once ThermAcryl is heated thoroughly it will coagulate, similar to the consistency of putty.
3. Carefully retrieve the hot ThermAcryl and knead it to ensure cohesion and place in desired tray. Push firmly onto arch.

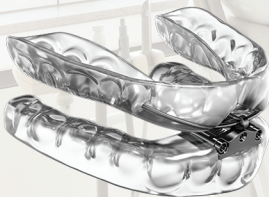

For more information, visit our website at www.oralartsdental.com/dreamtap

OralArts
DENTAL LABORATORIES HANTYVILLE, ALABAMA

| 800-354-2075