

Indication Guide for Metal Free Restorations

Products	Strength	Prep Design	Recommended Reduction	Minimum Reduction	Anterior Bridge	Posterior Bridge
Tetra-Z Zirconia	1,200 MPa	Slight Chamfer Margin	1.5 mm	0.6 mm Reduction	✓	✓ Max Pontic Span 25 mm
Element-Z Zirconia	1,050 MPa	Slight Chamfer Margin	1.5 mm	0.6 mm Reduction	✓	✓ Max Pontic Span 25 mm
Element-Z Lingual	1,050 MPa on biting surfaces	Slight Chamfer Margin	1.5 mm	0.6 mm Reduction	✓	✓ Max Pontic Span 25 mm
Ultra-Z Esthetic Zirconia	750 MPa	Chamfer Margin	1.5 mm	0.8 mm Reduction	✓ 1 Pontic	✓ No Molars / 1 Pontic
LiSi Press	508 MPa	Chamfer Margin	1.5 mm	0.8 mm Reduction	✓ 1 Pontic	✓ No Molars / 1 Pontic
Porcelain Fused to Zirconia	Forte 1,400 MPa	Chamfer Margin	1.5 mm	0.8 mm Reduction	✓	✓ 1 Pontic